


VERSUS APP USER GUIDE

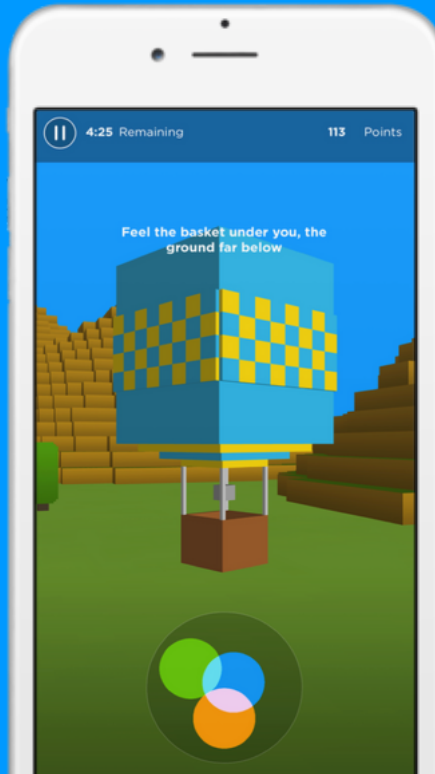

VERSUS

READY, SET, GO!

Find what you're looking for here.

03
GETTING STARTED

11
ASSESSMENT

23
EXERCISE

32
WARM UPS



36
PERFORMANCE

41
COMPARE

46
SETTINGS


51
LEARN MORE

VERSUS

GETTING STARTED

1

MAKE YOUR PURCHASE

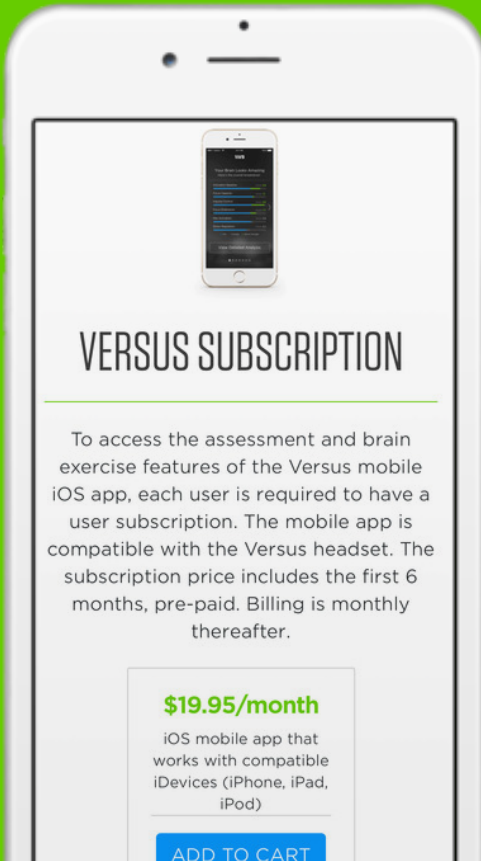


1. To access the NeuroPerformance Assessment (NPA) and full Versus exercises, you need an active Versus Subscription. If you are only interested in the assessment, you can purchase individual NPAs separate from the Versus Subscription.

2. Versus Subscriptions and individual NPAs are available for purchase at a discounted price at: shop.getversus.com. They can also be purchased in-app once you've created a Versus account.


2

DOWNLOAD THE APP

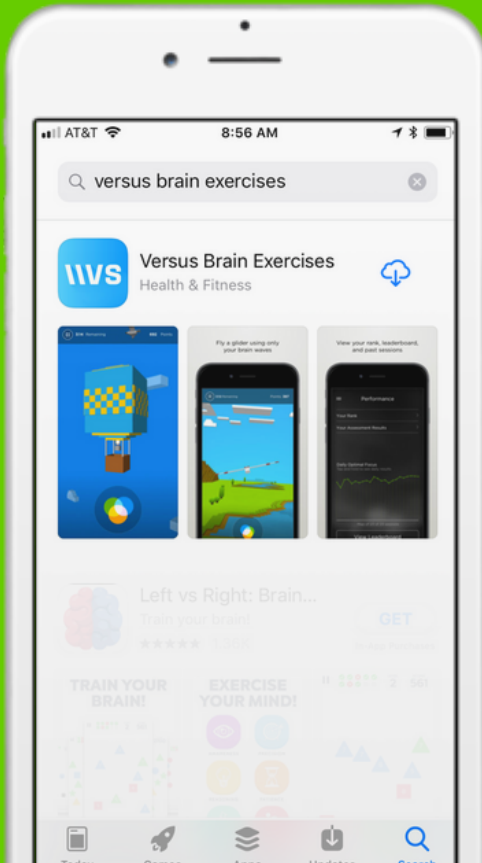

1. Search "Versus Brain Exercises" in the App Store on your compatible iOS device.*

2. If you are on an iPad, select the "iPhone Only" filter in the top left corner of the App Store.

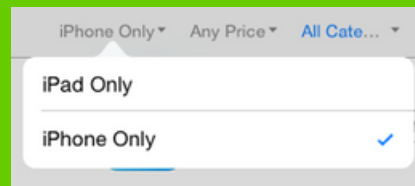

3. Download the Versus App onto your iDevice.

*Versus is compatible with: iPad Air & above, iPad Mini 2 & above, and iPhone 5, 5S, 6, 6+, 6S, SE, 7, 7+, 8, & X (iOS10 or later required).

3


OPEN THE APP



Once the Versus app has finished downloading, open it by double-clicking on the app icon on your mobile iOS device.

4

CREATE AN ACCOUNT

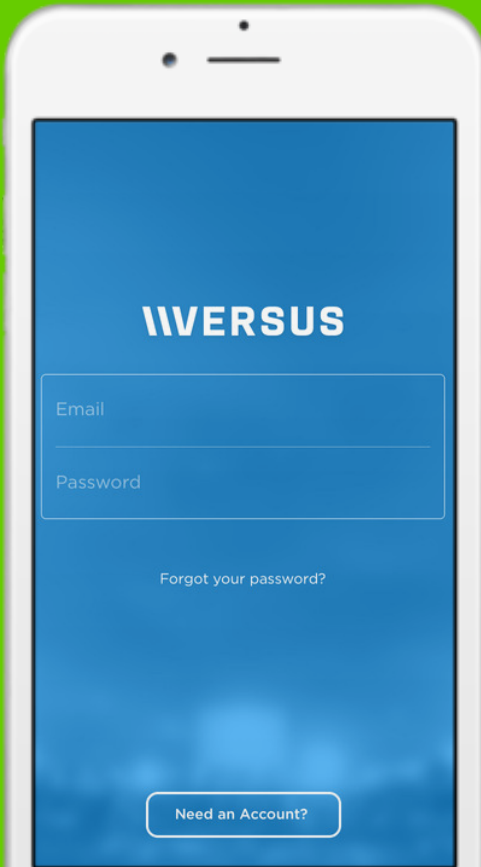


1. If you have not already registered a Versus account, click "Need an Account".

2. When you get to the Provider Team screen, enter your provider's team name. If you are not working with a provider or do not wish to join a team, press "Skip".

5

SIGN IN TO THE APP

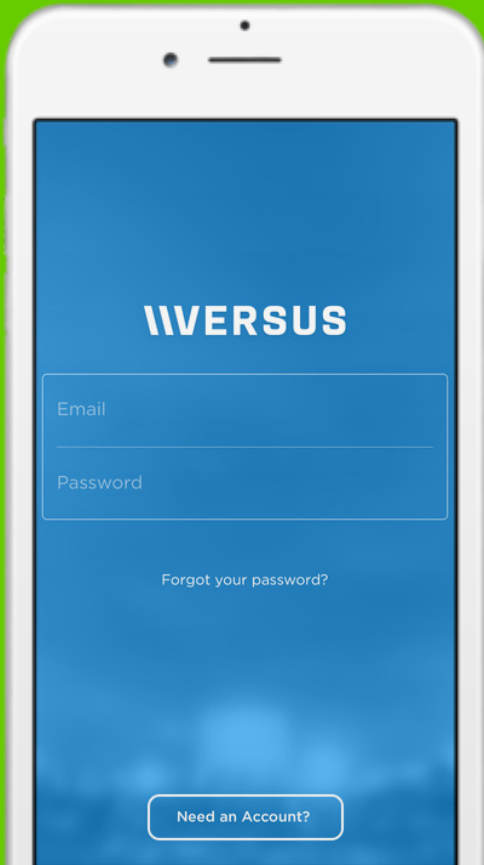


Using the email address and password you registered, log in to the Versus app.

6

UPLOAD AN AVATAR


You are not required to upload an avatar.
Your avatar will be visible to other Versus users.

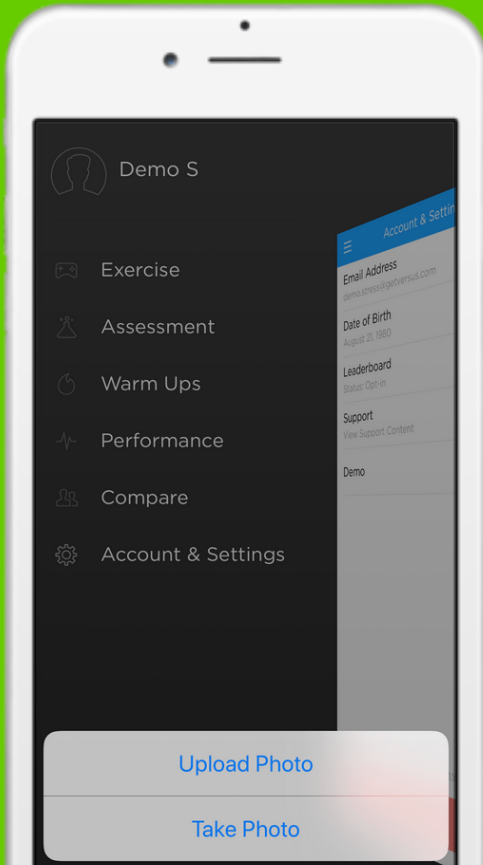

1. Open the Navigation Pane by pressing the menu button in the top left corner.




2. In the top left corner of the Navigation Pane, select the avatar button.

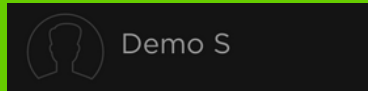

3. Click "Upload Photo" to use a picture from your device as your avatar. Click "Take Photo" to take a picture to use as your avatar.

7

START NAVIGATING

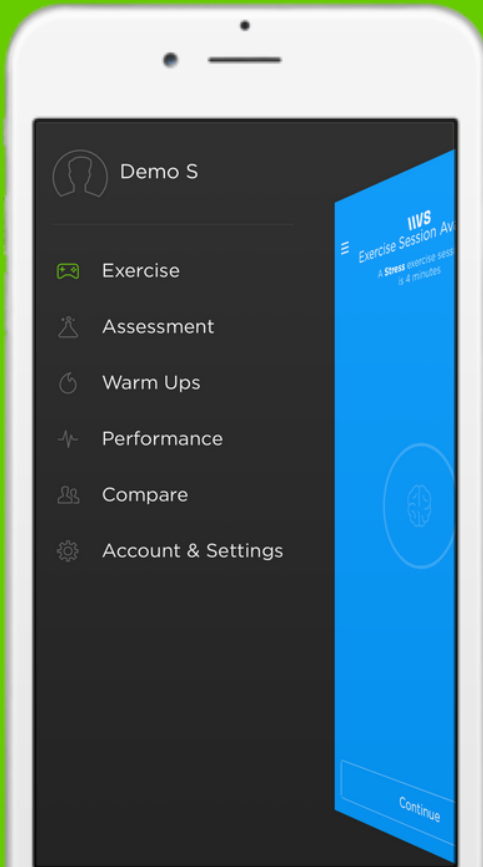


1. Open the Navigation Pane by pressing the menu button in the top left corner.

2. Access the tabs listed below by selecting them from the Navigation Pane.

- Exercise: Access the Versus exercises.
- Assessment: Complete assessments, as available.
- Warm-Ups: Complete practice exercises.
- Performance: View your exercise and assessment results.
- Compare: Compare your performance to other users.
- Account & Settings: View and edit your account settings.

ASSESSMENT

NEUROPERFORMANCE ASSESSMENT (NPA)

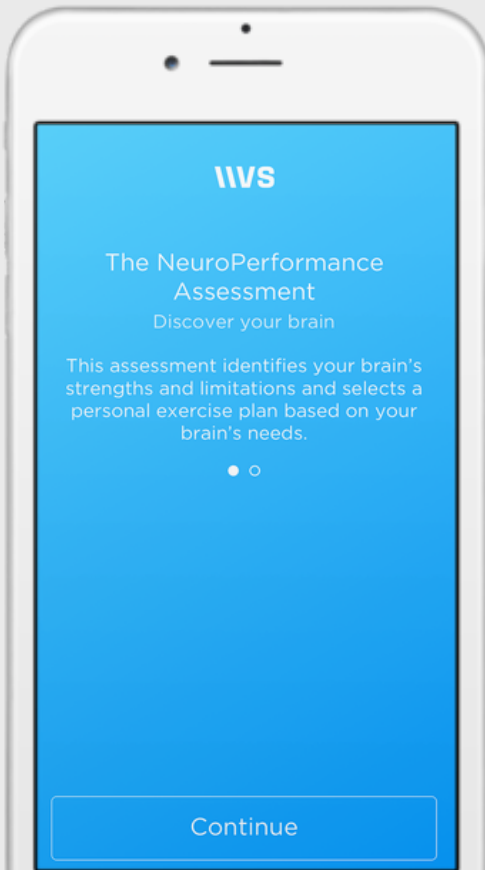


The NPA is used to identify your individual strengths and weaknesses around brain performance. It consists of two tasks: an eyes-closed (EC) task and a continuous performance task (CPT). You must complete an initial NPA before you can start the Versus exercises.

Duration: 14-22 Minutes

Purpose: Based on the results of the NPA, the app will automatically assign you the most appropriate Performance Protocol for your exercises. On a standard plan, you'll complete ~8 hours of exercise on that Protocol. You can then take another NPA to assess your progress.

EYES-CLOSED (EC) TASK

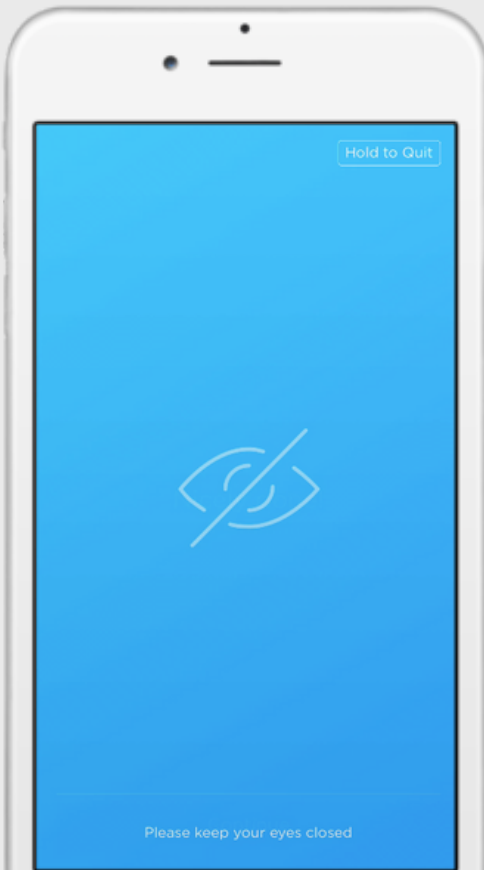


The eyes-closed task is used to get a baseline reading for your brain activity - to see what your brain looks like when nothing is being demanded of it.

Duration: 2-10 Minutes

Instructions: When instructed, close your eyes and keep them closed until you hear a three tone sound. Gentle tones will play throughout the task to let you know it's ongoing, so make sure your audio cable is connected. Relax and remain calm. The task will continue for 10 minutes, or until the app has collected 2 full minutes of quality data.

CONTINUOUS PERFORMANCE TASK (CPT)

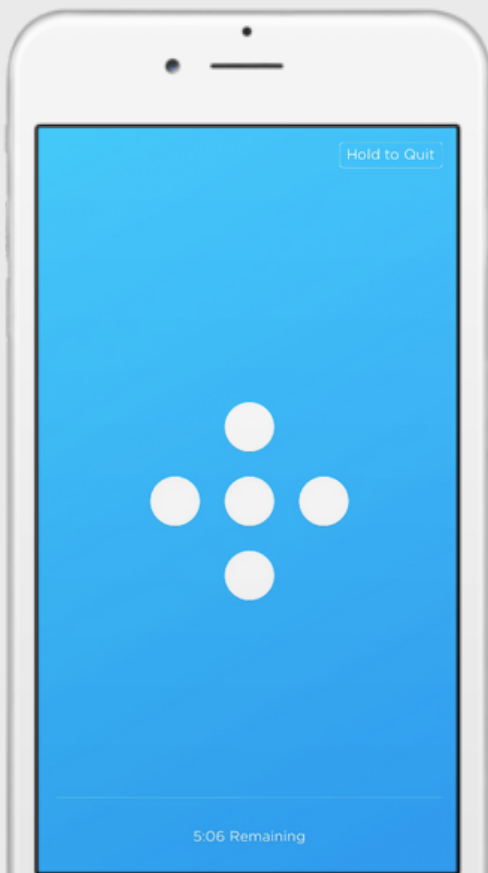

The CPT is used to determine how your brain functions over an extended task with varying levels of difficulty.


Duration: 12 Min. (Practice + Task)

Instructions: Before starting the CPT, you must complete the practice CPT (1 min). In both, you'll be shown a series of targets and non-targets. Tap the screen once, using a single finger, only when the specific target is shown.


Versus Tip: You must complete the entire CPT at one time. If the CPT is interrupted, you'll have to start over. Complete in a distraction-free environment and remain still to avoid sensor disconnection.

1 ACCESS THE ASSESSMENT

1. From the Navigation Pane, select the Assessment tab.

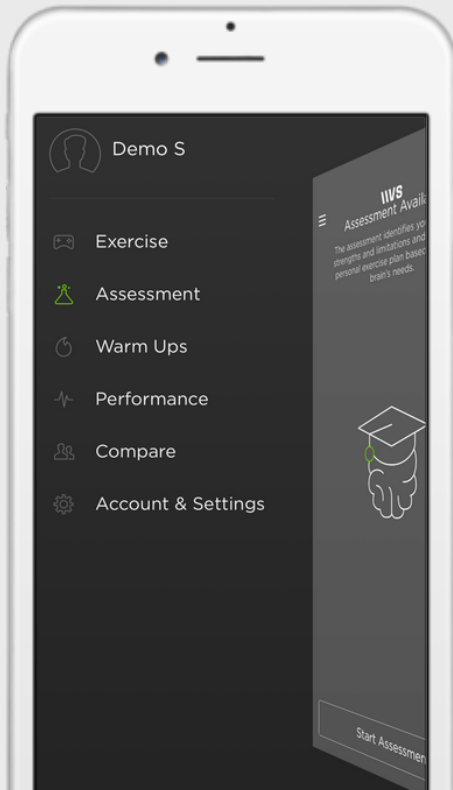



2. In the Assessment screen, press "Start Assessment".

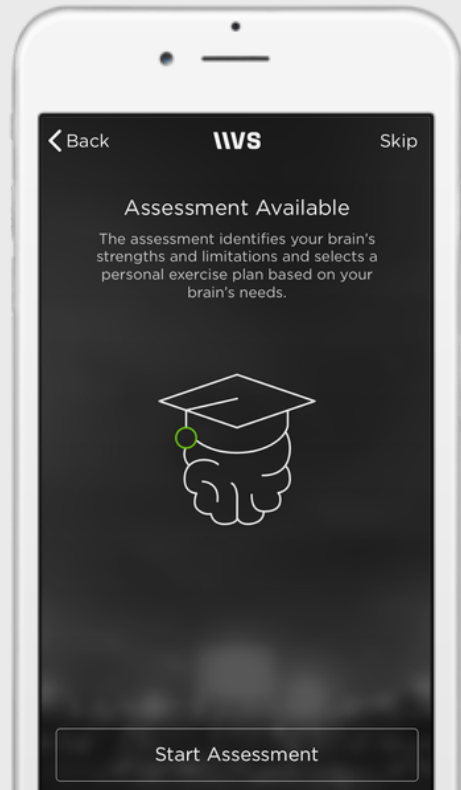

2

VIEW THE INSTRUCTIONS

Carefully read the instructions and watch the instructional video. Press the "Continue" button when you are ready to proceed.

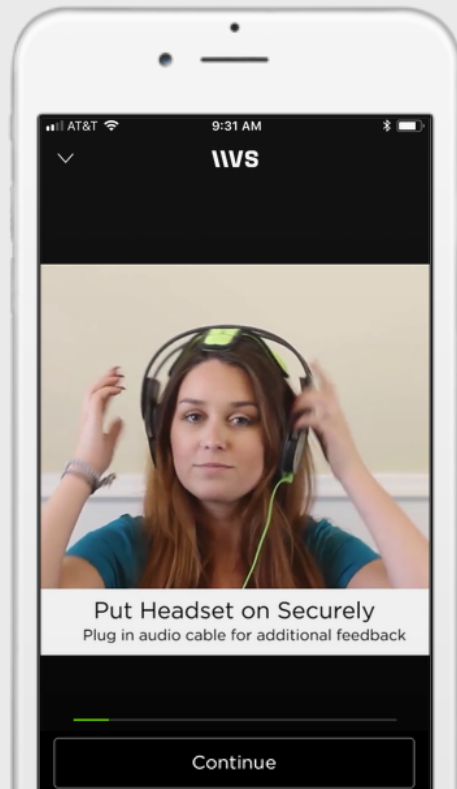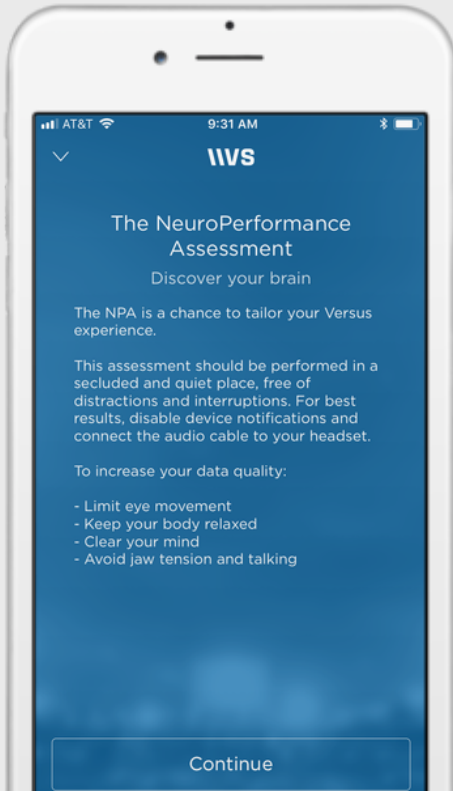



3

GET CONNECTED

Need help? Visit support.getversus.com

1. From the list, choose the serial number of your headset (located on the right interior).

2. Adjust the headset until all 5 circles, which represent the sensors, are green.

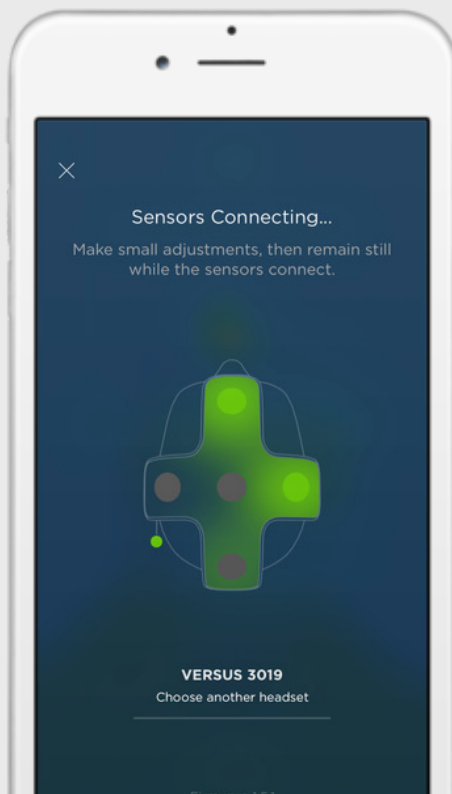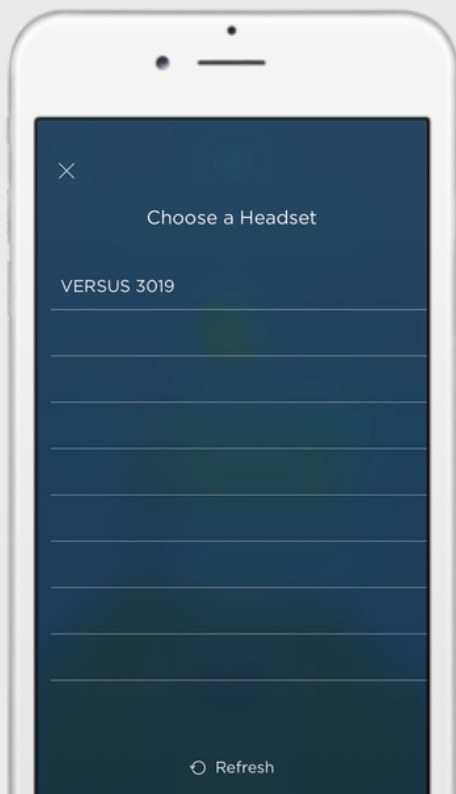


4

LEARN ABOUT ARTIFACT

1. Learn how to avoid artifacts. Artifacts are data points that reflect movement (tension, blinking, etc.) instead of true brain activity.

2. Practice minimizing movement to reduce artifacts. Keep the line “calm” and tap the Continue button when it appears to proceed

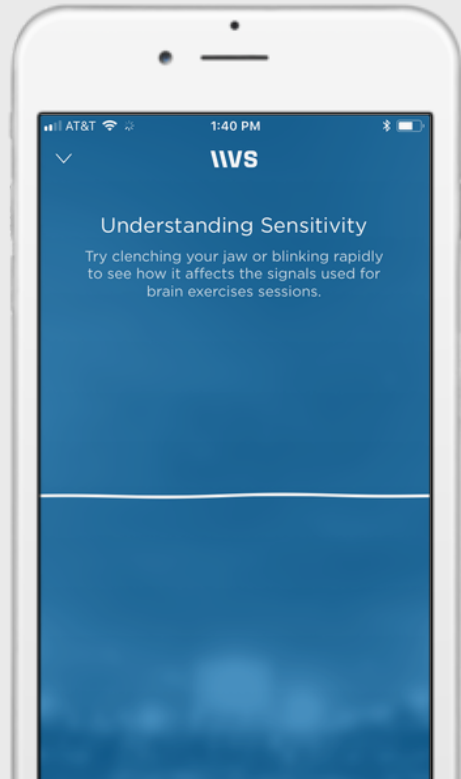


5

COMPLETE THE EC TASK

1. When prompted by the app, close your eyes and keep your mind clear and body relaxed.

2. When you hear a 3 tone ring, open your eyes. The eyes-closed task is complete.


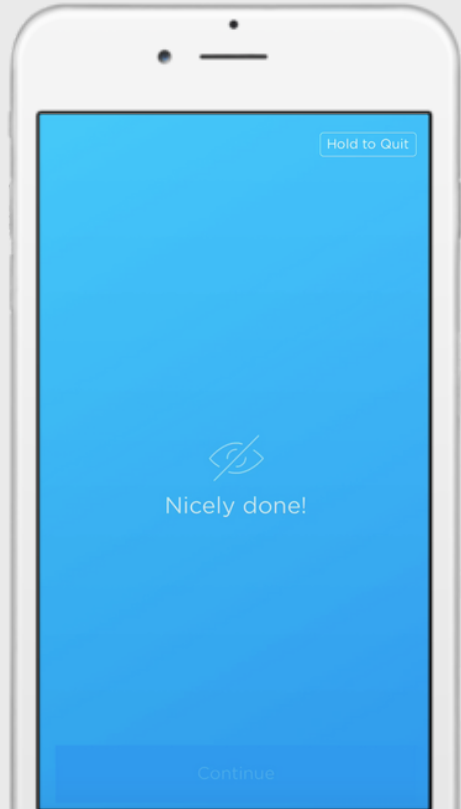


6

COMPLETE THE CPT

1. Read the instructions and complete the practice Continuous Performance Task.

2. Only tap the screen when you see the target. Avoid excess blinking and movement.

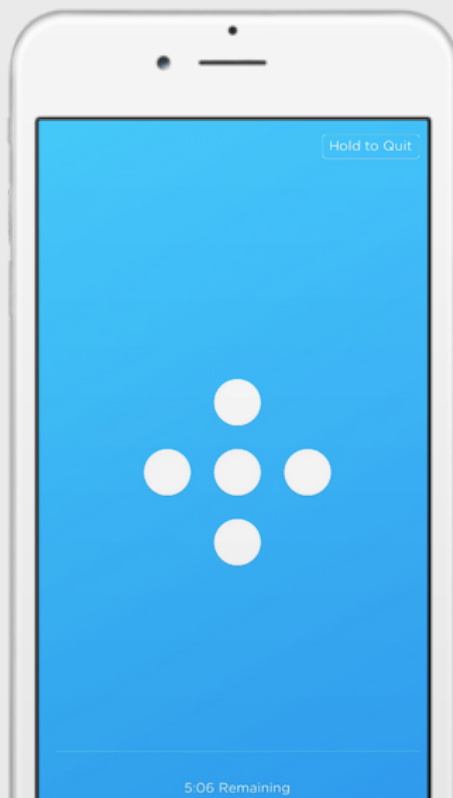
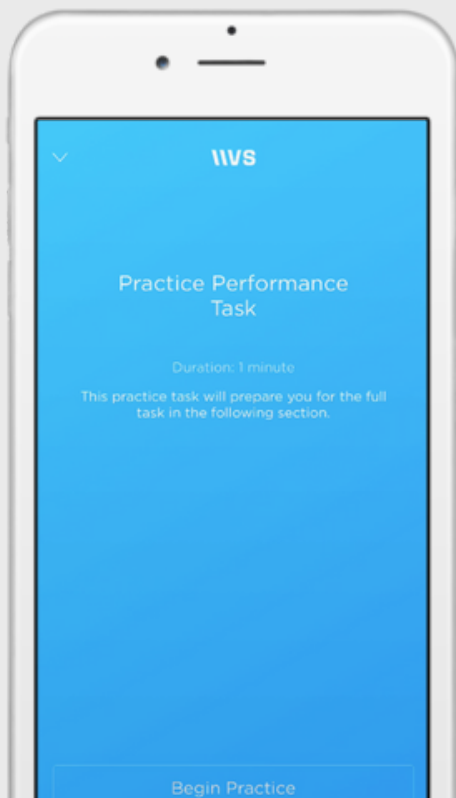


7

VIEW YOUR RESULTS

1. View your construct scores.
These results will also be automatically emailed to you.

2. Click on the individual constructs to view a detailed explanation.

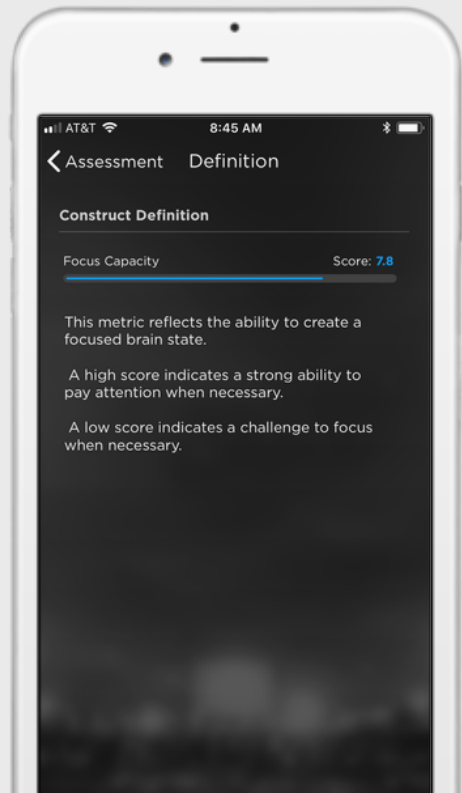
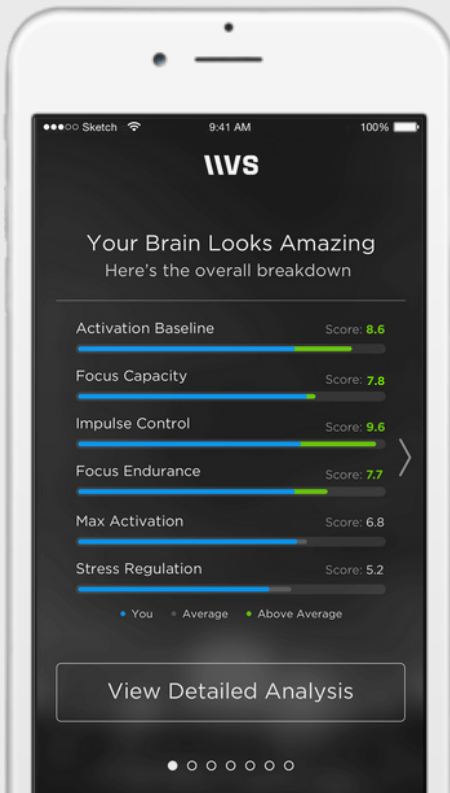


8

VIEW DETAILED ANALYSIS

1. Click "View Detailed Analysis" to view an interpretation of your assessment results, which will also be emailed to you.

2. After reading your results interpretation, press "Continue" to view details about your assigned Performance Protocol.

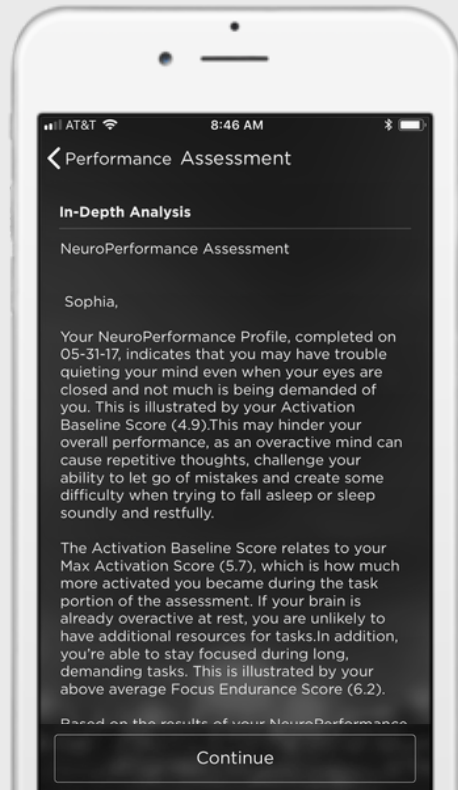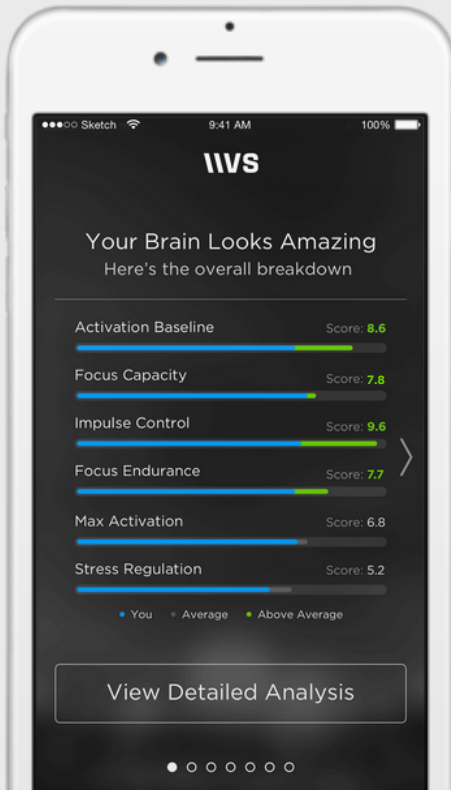

VERSUS

EXERCISE


EXERCISE



From the Exercise tab, you can complete the Versus exercises and available assessments.

To get more familiar with the Versus system before you start the exercises, try the Versus warm ups, available in the Warm Ups tab on the Navigation Pane.


1 ACCESS THE EXERCISES

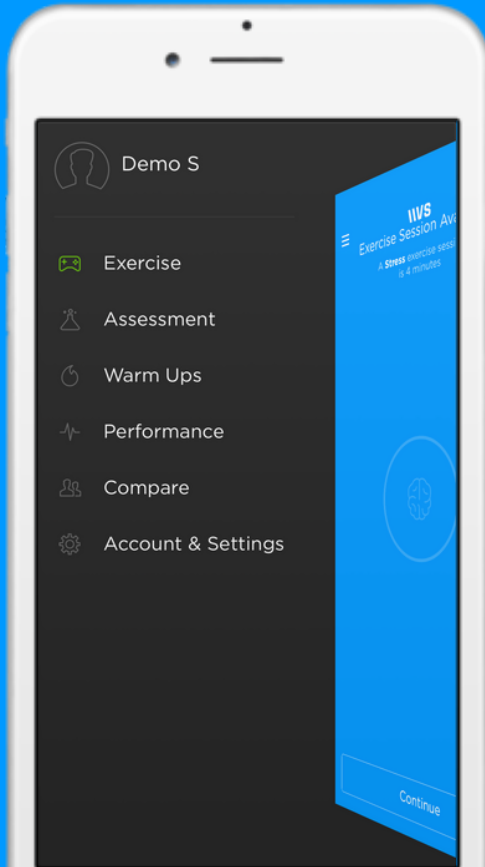

From the Navigation Pane, select the Exercise tab.

2

START THE EXERCISES

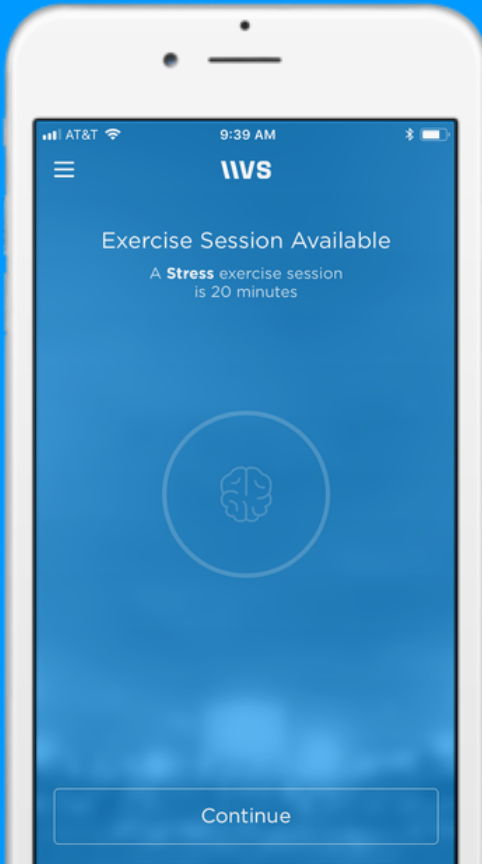


1. Read the session description. You will either be on a Stress or Focus protocol.

2. Press "Continue" to proceed.

3

READ THE INSTRUCTIONS

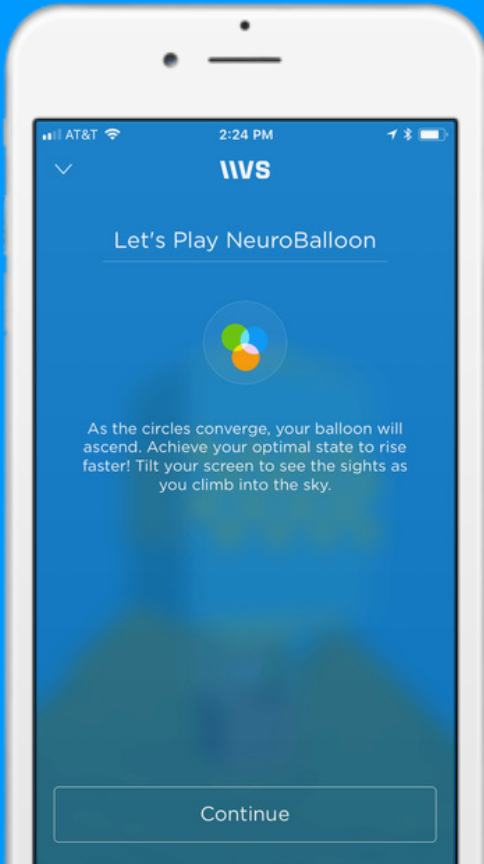

1. Before each of the 4 games, you'll be provided with instructions on how to play. Read these instructions carefully.
2. Press "Continue" to proceed.



4

GET CONNECTED

Need help? Visit support.getversus.com

1. From the list, choose the serial number of your headset (located on the right interior).

2. Adjust the headset until all 5 circles, which represent the sensors, are green.

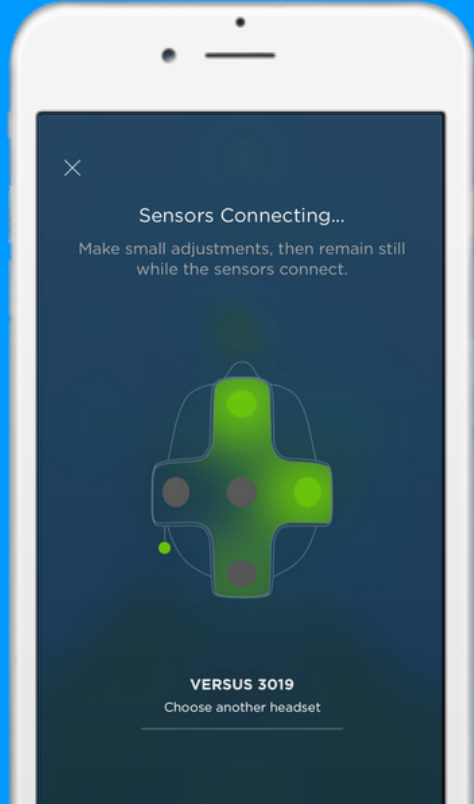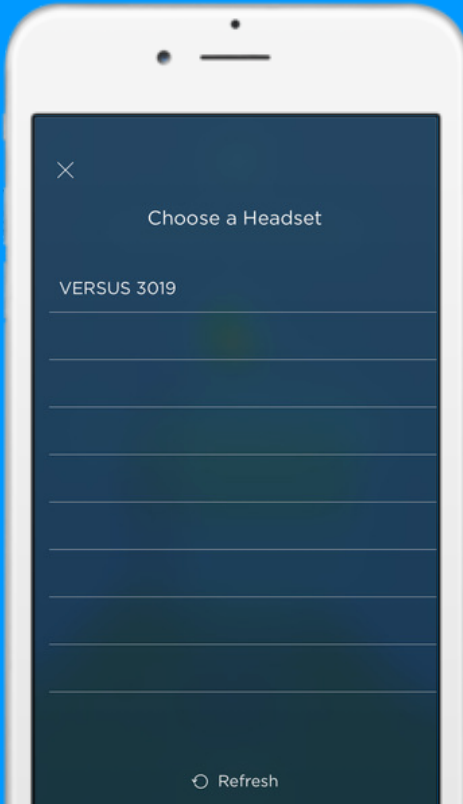

5

COMPLETE THE EXERCISES




The app will guide you through 4 games, each the same length of time.

The primary goal of each of the games is to get the 3 circles at the bottom of the screen to converge by creating the desired brain state.

The secondary goal is to engage with the game and earn Neupoints. These points are accumulated as you bring the three circles together and do well in the games.

6

VIEW YOUR ZONE TIME

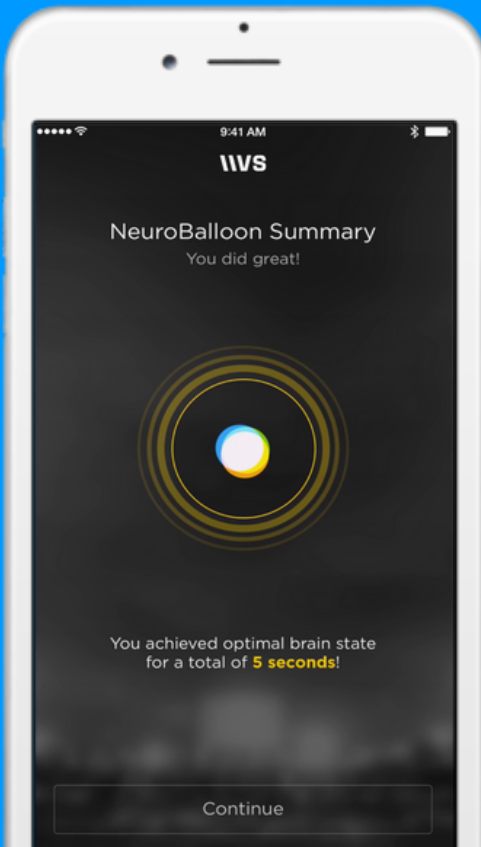


At the end of each game, you'll be shown your Points and Optimal Zone Time for that game. When you complete an exercise session, you'll be shown your total Points and Optimal Zone Time for the entire session.

Optimal Zone Time: This is the amount of time you were in the optimal zone (all 3 circles were converged) for the desired brain state. The optimal zone is difficult to reach; you should not expect a significant amount of time. If you don't see this screen, it means you didn't accumulate any Optimal Zone Time.

7

VIEW YOUR POINTS

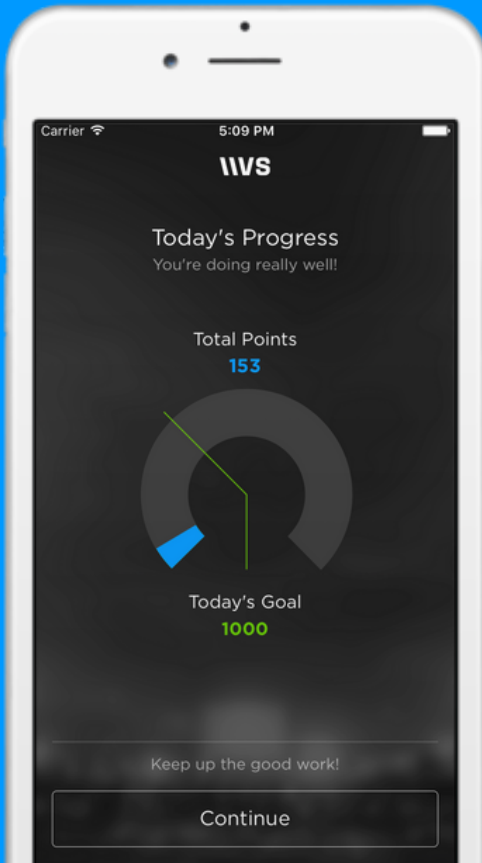


Points: The more you are able to get the circles to converge, the more points you'll earn. You'll earn points more quickly when you're in the optimal zone.

Points Goal: Your Points Goal is based on how well you did in your previous exercise session.

VERSUS

WARM UPS


WARM UPS

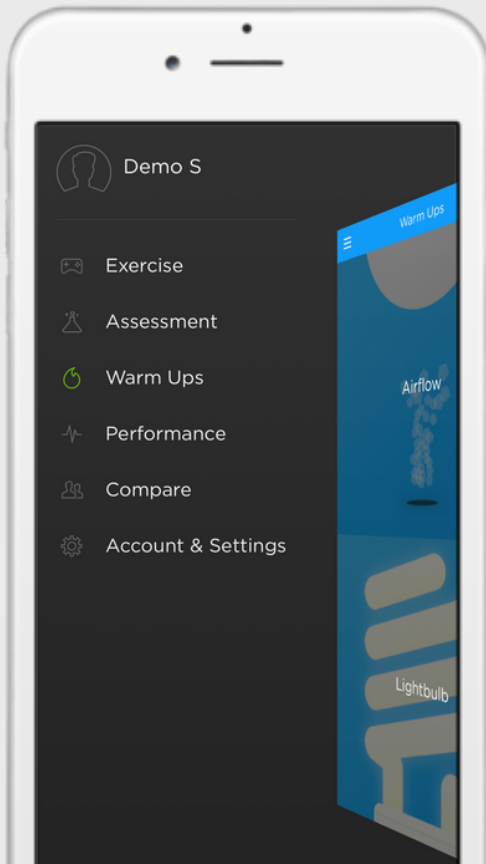

Warm ups are practice exercises that can help you get accustomed to using Versus.

To access the warm ups, select the Warm Ups tab from the Navigation Pane.

AIRFLOW

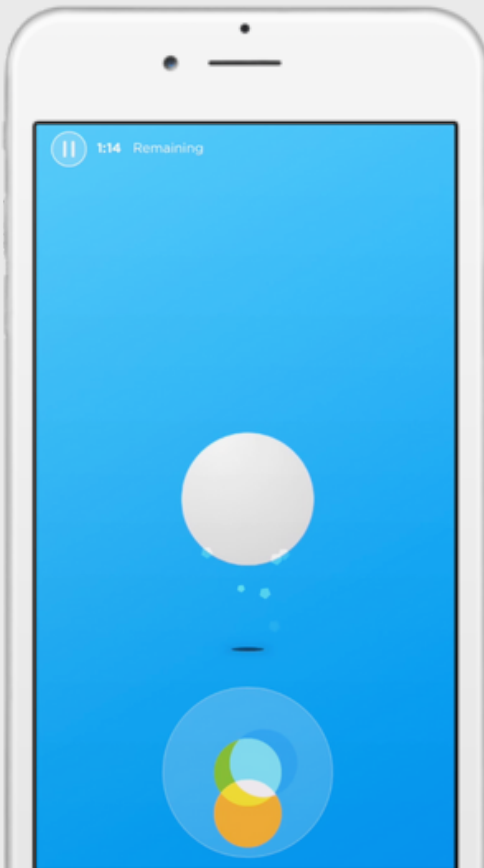

Instructions: Read the provided instructions and press "Continue" to proceed. Complete the sensor verification process, as you would for the standard Versus exercises.

Goal: Get the 3 circles at the bottom of the screen to converge by creating the desired brain state. The more effectively you produce that state, the higher the ball will rise.

LIGHTBULB




Instructions: Read the provided instructions and press "Continue" to proceed. Complete the sensor verification process, as you would for the standard Versus exercises.

Goal: Get the 3 circles at the bottom of the screen to converge by creating the desired brain state. The more effectively you produce that state, the more the lightbulb will illuminate.

VERSUS

PERFORMANCE


PERFORMANCE

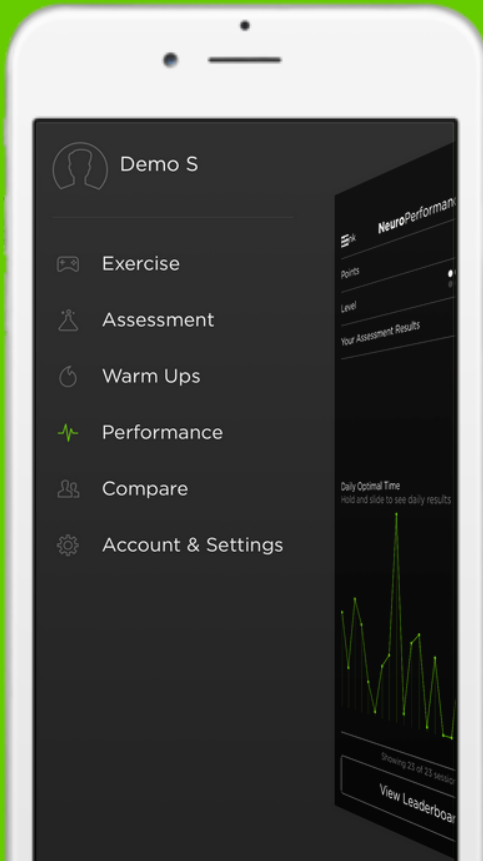

In the Performance tab, you can view the results of your past assessments and exercise sessions.

To access the Performance tab, select "Performance" from the Navigation Panel.

PERFORMANCE

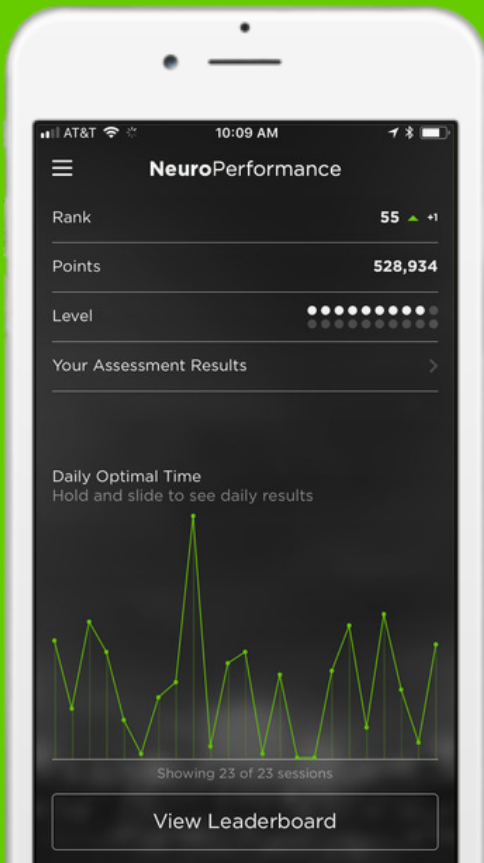

Rank: Your position in relation to other Versus users, based on your points total.

Points: The total amount of points you have accumulated across all your exercise sessions.



Level: Your current game level in the exercises.

Your Assessment Results: Click to view the results of your most recent NPA.

DAILY STATISTICS

1. Press and hold the green dots under Daily Optimal Time to see the breakdown of the results of each session.


2. View the Points and Optimal Zone Time you accumulated in the exercise session on the selected date.

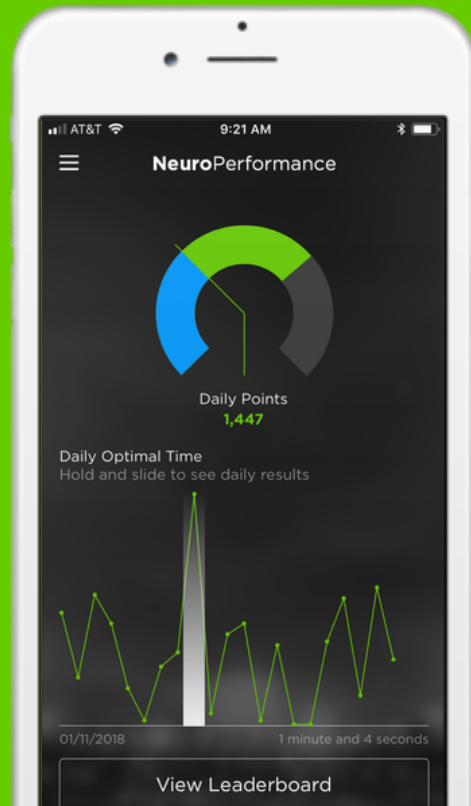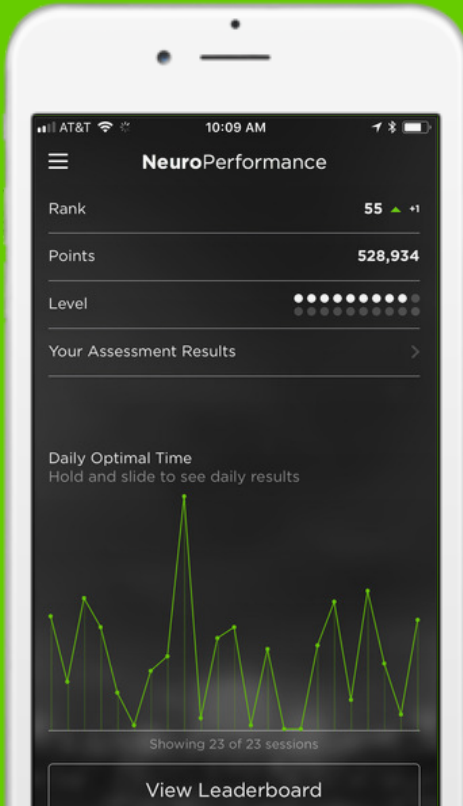


LEADERBOARD

Compare your rank with other Versus users. You are automatically opted-in to the Leaderboard. To opt-out, go to Account & Settings.

1. Press "View Leaderboard"
at the bottom of the screen.

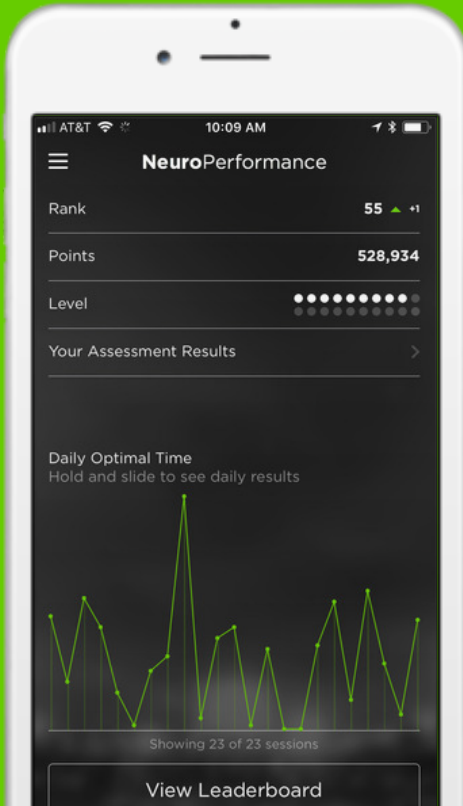


2. Press "Jump to Your
Position" to view your rank.

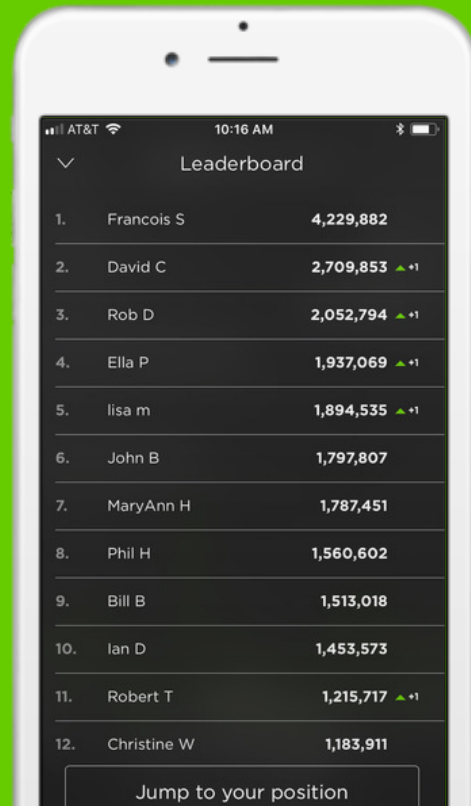

VVERSUS

COMPARE


COMPARE

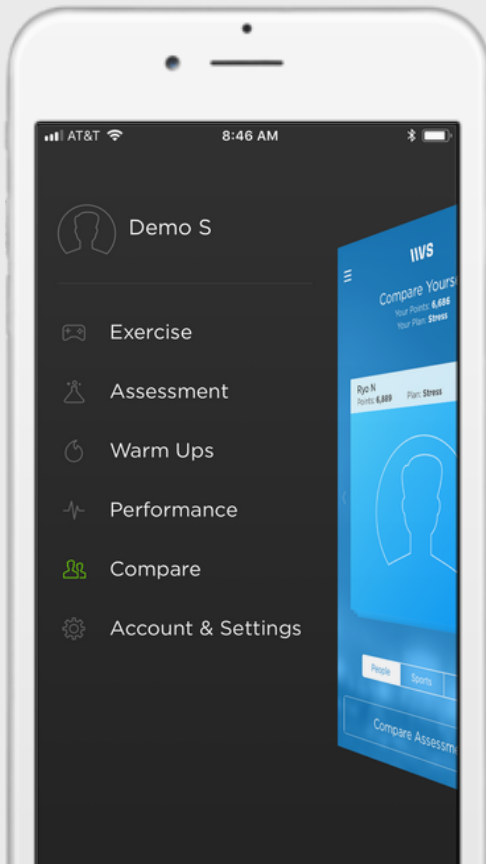

In the Compare tab, you can compare your exercise and assessment results with those of other Versus users.

To access the Compare tab, select "Compare" from the Navigation Panel.

COMPARE TO USERS

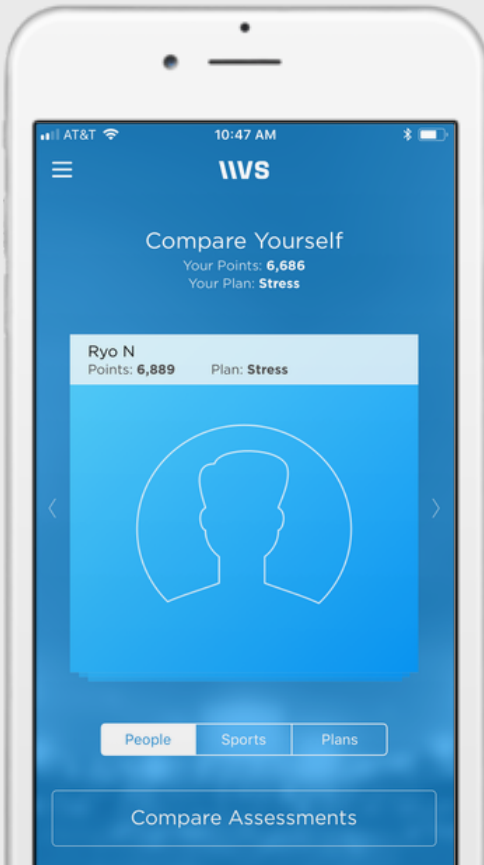


1. To compare yourself to individual Versus users, press "People".

2. Swipe right or left to see different users.

3. Select "Compare Assessments" to compare your NPA results to the selected user. Click on the individual constructs to view more details.

COMPARE TO SPORTS

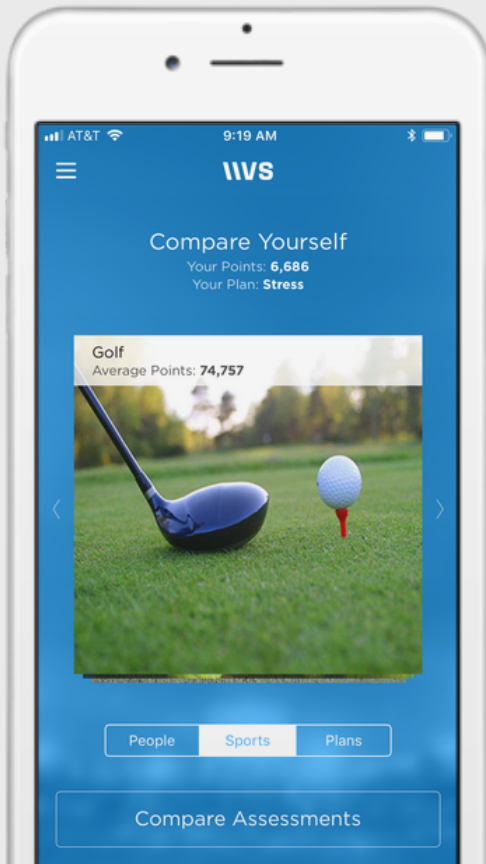


1. To compare yourself to the average for Versus users registered under a particular sport, press "Sports".

2. Swipe right or left to see different sports.

3. Select "Compare Assessments" to compare your NPA results to the average for users in the selected sport. Click on the individual constructs to view more details.

COMPARE TO PLANS

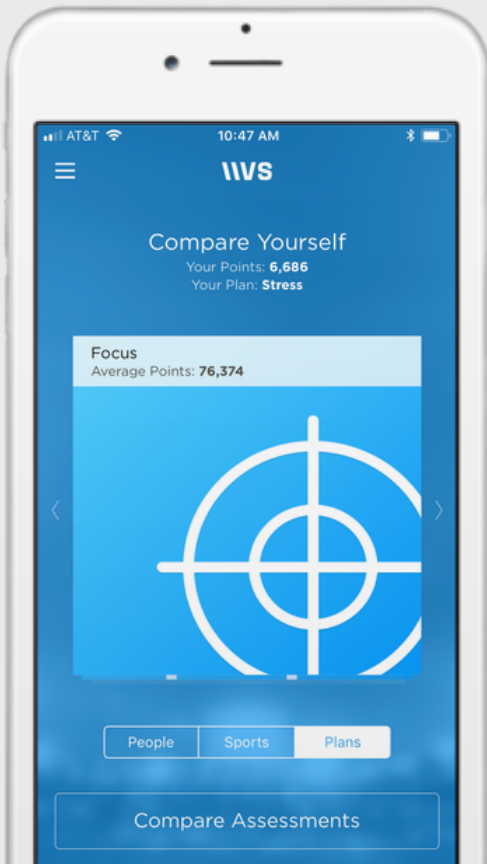

1. To compare yourself to the average for Versus users on Stress or Focus plans, press "Plans".


2. Swipe right or left to see different plans.

3. Select "Compare Assessments" to compare your NPA results to the average for users on the selected plan. Click on the individual constructs to view more details.

VERSUS

ACCOUNT & SETTINGS

ACCOUNT & SETTINGS

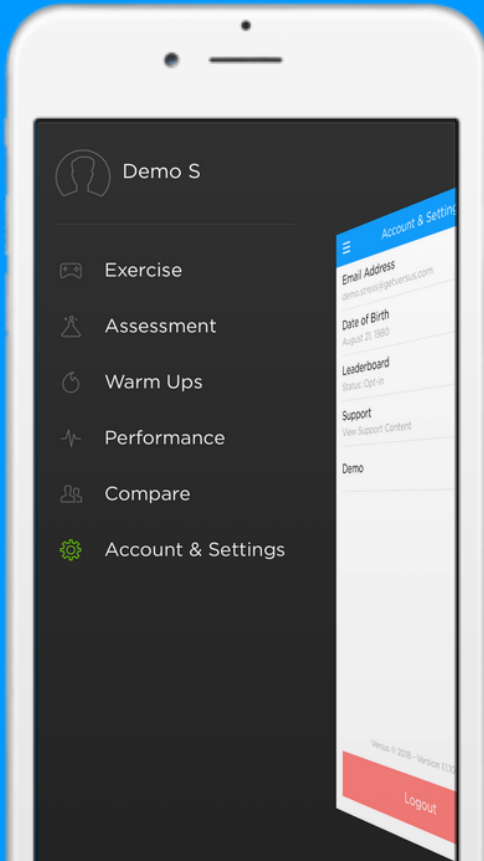


From the Account & Settings tab, you can view and edit your account settings and access support materials.

To access this section, select "Account & Settings" from the Navigation Panel.

ACCOUNT & SETTINGS

Your email address and date of birth cannot be changed in-app.
To modify these, please contact support@getversus.com

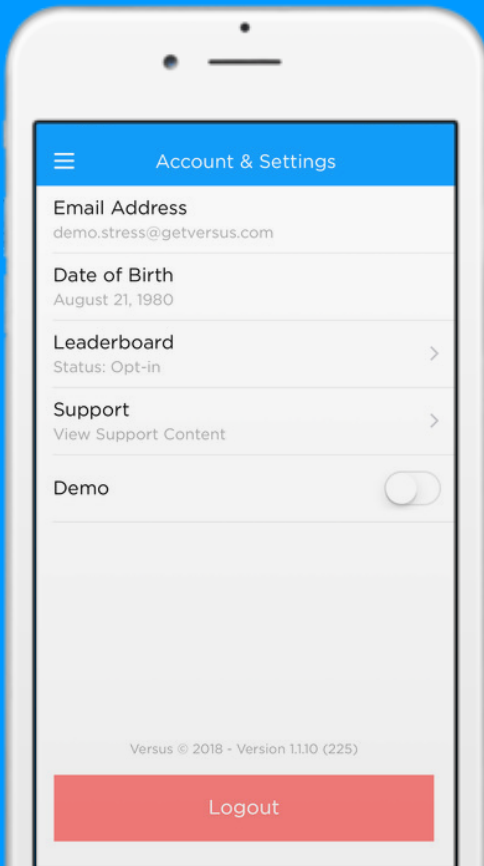


Leaderboard: Select to opt in or out of the Leaderboard.

Support: Select to access support videos and information.

Logout: Select to log out of your Versus account.


LEADERBOARD SETTINGS

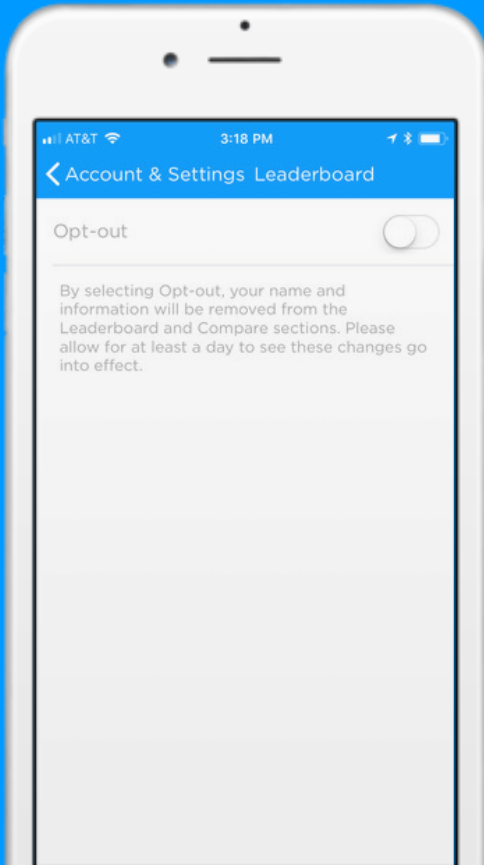

By default, you will be opted in to the Leaderboard. This means that other Versus users will be able to see your first name and last initial, avatar, current plan, and points total in the Leaderboard and Compare sections.

Opt-Out: To opt-out of the Leaderboard, click the toggle button so that it is in the off position. Your information will no longer be visible in the Leaderboard or Compare sections.

SUPPORT

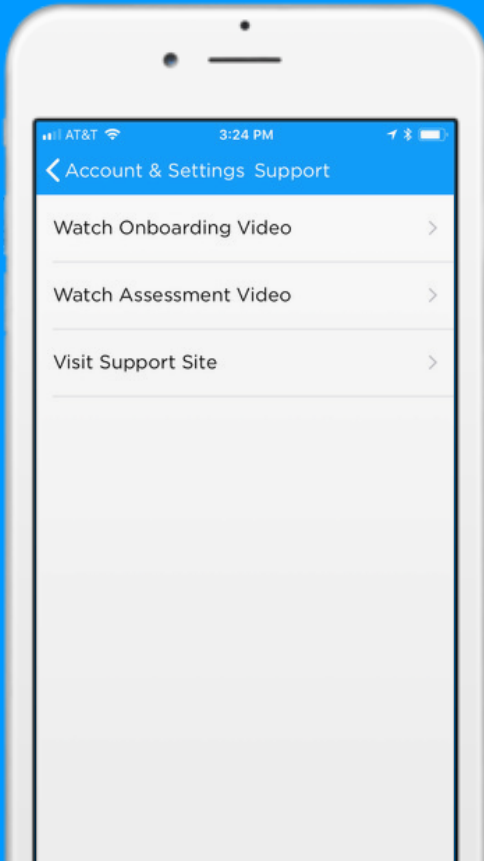

Onboarding Video: Select to watch a video about connecting the Versus headset.

Assessment Video: Select to watch a video about completing the assessment.

Support Site: Select to visit the Versus support site (support.getversus.com).

LEARN MORE


Questions? Comments? We're here to help!



[getversus](https://www.facebook.com/getversus)



[@getversus](https://twitter.com/getversus)

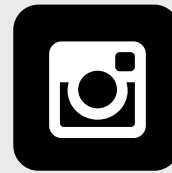

[@get.versus](https://www.instagram.com/get.versus)



support@getversus.com



[getversus.com](https://www.getversus.com)



support.getversus.com